

Top Ten Things
You and Your Student Should Know
About the Braille Sense Plus

NOTE: This worksheet is intended to be used with the Introduction to the Braille Sense Plus training. The audio for the training can be found at www.gwmicro.com/braillesense. Then, click on the link called Online Training Session 1.

1. For sighted teachers, a visual diagram of the Braille Sense Plus can be found at
http://www.gwmicro.com/images/braille_sense/braille_sense_plus.jpg
2. You can adjust the voice with the following keystrokes:
 - a. BACKSPACE-F1 will decrease the volume. BACKSPACE-F4 will increase the volume.
 - b. SPACE-F1 will decrease the rate. SPACE-F4 will increase the rate.
 - c. ENTER-F1 will decrease the tone. ENTER-F4 will increase the tone.
3. You can turn the speech on or off by pressing BACKSPACE-F2.
4. You can toggle the LCD display on or off by pressing DOTS 4-5-6-ENTER.
5. You can access the Program menu at any time by pressing F1.
6. If you are in an application, and you want to access the menu for that application, pressing F2 or SPACE-M will open the menu.
7. You can access the Options menu at any time by pressing SPACE-O.

- a. Once in the Options menu, you can change any of the options by pressing SPACE or BACKSPACE.
 - b. You must press ENTER in the Options menu to save the changes.
8. You can toggle One-handed Mode on or off by pressing BACKSPACE-1-2-3.
9. To open an application, press ENTER on it.
10. When in the File Manager, press ENTER on a drive to open it. To get out of the list of folders in the drive, press BACKSPACE.